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Storage between 2°C-27°C/35°F-80°F

Light Sensitive

Warning: Acrylate monomers are known to produce allergic reactions in certain sensitive individuals. May cause eye and skin irritation.

Precautions: Avoid eye and skin contact. Wear gloves when handling this material.

First Aid:
Eye contact: Immediately flush with plenty of water. See a physician if irritation persists.
Skin contact: Wash affected area with soap and water. See a physician if irritation persists.

- To insert capsule into dispenser, open dispenser handle as wide as possible. Pull plunger toward opened handle.

- Select one of the following options.

Option A: Remove the cap from the capsule. Place capsule tip into the barrel slot (Figure 1). Seat the capsule in the barrel by pushing the capsule forward (away from the handle) as far as possible (Figure 2). Push plunger toward capsule. Close handle grip until plunger contacts capsule.

Option B: Place the capsule upside down into the barrel slot with the cap on. Push the capsule forward (away from the handle) as far as possible, then rotate the capsule to the correct orientation in the dispensing gun. Push plunger toward capsule. Close handle grip until plunger contacts capsule. Remove the cap from the capsule.

- Dispense the material** onto the bracket base using a slow, steady pressure. See Figure 3.

- Remove the used capsule.** Open the dispenser handle as wide as possible. Pull plunger away from capsule. Push the capsule toward the plunger. Turn dispenser upside down to drop capsule into hand. See Figure 4.

Disinfection Information: Please refer to literature 011-650, “Reprocessing Instructions for Use”.

<div> <div></div> <div>3M Unitek</div> </div> <div> <div> <div></div> <div>Instructions For Use</div> </div> <div> <div></div> <div>Mode d'emploi</div> </div> <div> <div></div> <div>Gebrauchsanleitung</div> </div> <div> <div></div> <div>Istruzioni per l'uso</div> </div> <div> <div></div> <div>Instrucciones de uso</div> </div> <div> <div></div> <div>Gebruiksaanwijzing</div> </div> </div>	<div> <div></div> <div>Brugsanvisning</div> </div> <div> <div></div> <div>Käyttöohjeet</div> </div> <div> <div></div> <div>Οδηγίες Χρήσης</div> </div> <div> <div></div> <div>Instruções de utilização</div> </div> <div> <div></div> <div>Brugsanvisning</div> </div>
--	--

<div> <div></div> <div>3M</div> </div> <div> <div> <div></div> <div>3M Unitek</div> </div> <div> <div></div> <div>Orthodontic Products</div> </div> <div> <div></div> <div>2724 South Peck Road</div> </div> <div> <div></div> <div>Monrovia, CA 91016 USA</div> </div> </div>	<div> <div></div> <div>CE</div> </div> <div> <div></div> <div>REP</div> </div> <div> <div></div> <div>0086</div> </div> <div> <div></div> <div>3M Deutschland GmbH</div> </div> <div> <div></div> <div>Orthodontic Products</div> </div> <div> <div></div> <div>Carl-Schurz-Str. 1</div> </div> <div> <div></div> <div>D-41453 Neuss, Germany</div> </div> <div> <div></div> <div>+49(0)8191/ 9474-0</div> </div>
---	--